


Washing and Drying vinyl-backed

Repositioning Pads and Wheelchair Pad Covers

Our Repositioning Pads and Wheelchair Pad Covers are of vinyl-backed terrycloth. Yours may be Navy Blue, Hunter Green, or both. You can help your facility get good value from these items by carefully following these laundering instructions.

Washing Instructions Developed In Cooperation With Ecolab, Inc.

	Operation	Water Level	Temp.	Time (Min.)	Supplies for 100 lb. Rated Equipment	Notes
1.	Suds/Bleach	Low	140° -160°	6-8	8-14 oz. Tri-Star L-2000 2-4 oz. Oxy-Brite or other oxygen bleach	 Chlorine Bleach
2.	Rinse	High	100° -120°	3		
3.	Extract			2		Optional extract to improve rinsing
4.	Rinse	High	100° -120°	3		
5.	Rinse	High	100° -120°	3		
6.	Final Rinse	Low	100° -120°	4	2-3 oz. Aqua Soft or Solid Soft	Optional to reduce static
7.	Dry		150° maximum until just dry, then use cool-down cycle			

Recommendations may vary, depending upon soil levels, water hardness, and your equipment. Your local Ecolab specialist can design a wash formula specifically for your conditions.

Alternate detergents: Tri-Star Glo, Tri-Star Vector, Tri-Star Re-4000, Eco-Line Vector 2, Solid Brite, Solid Star, Solid Surge.



1. **DON'T USE CHOLORINE BLEACH. NONE! NEVER!**

Use oxygenated "All fabric" bleach, but only when necessary. Use as little as possible, too.



2. **DRY ITEMS AT 150° Or Lower**

They'll still dry quickly, and you'll be helping make them last even longer. Don't overdry, and use the cool-down cycle after the drying cycle.

BLEACHING AND HIGH HEAT DRYING will harden and crack the vinyl waterproof layer of these items.

AND it will **void** Powderhorn's guarantee. **Please don't!**

Questions?

Problems?

Suggestions?

Call 1-800-336-1414

Powderhorn Industries, 931 North Park Ave., P.O. Box 1443, Montrose, CO 81402