


Washing and Drying vinyl-backed Wheelchair Pad Covers

Our Wheelchair Pad Covers are of the same vinyl-backed terrycloth as our StainHider bibs. Yours may be Navy Blue or Hunter Green. You can help your facility get good value from these items by carefully following these laundering instructions.

Washing Instructions Developed In Cooperation With Ecolab, Inc.

| Operation | Water Level | Temp. | Time (Min.) | Supplies for 100 lb. Rated Equipment | Notes |
|----------------|-------------|---|-------------|--|---|
| 1. Suds/Bleach | Low | 140° -160° | 6-8 | 8-14 oz. Tri-Star L-2000 2-4 oz. Oxy-Brite or other oxygen bleach |  Chlorine Bleach |
| 2. Rinse | High | 100° -120° | 3 | | |
| 3. Extract | | | 2 | | Optional extract to improve rinsing |
| 4. Rinse | High | 100° -120° | 3 | | |
| 5. Rinse | High | 100° -120° | 3 | | |
| 6. Final Rinse | Low | 100° -120° | 4 | 2-3 oz. Aqua Soft or Solid Soft | Optional to reduce static |
| 7. Dry | | 150° maximum until just dry, then use cool-down cycle | | | |

Recommendations may vary, depending upon soil levels, water hardness, and your equipment. Your local Ecolab specialist can design a wash formula specifically for your conditions.

Alternate detergents: Tri-Star Glo, Tri-Star Vector, Tri-Star Re-4000, Eco-Line Vector 2, Solid Brite, Solid Star, Solid Surge.



1. DON'T USE CHOLORINE BLEACH. NONE! NEVER!

Use oxygenated "All fabric" bleach, but only when necessary. Use as little as possible, too.



2. DRY BIBS AT 150° Or Lower

They'll still dry quickly, and you'll be helping make them last even longer. Don't overdry, and use the cool-down cycle after the drying cycle.

BLEACHING AND HIGH HEAT DRYING will harden and crack the vinyl waterproof layer of the Wheelchair Pad Covers.
AND it will **void** Powderhorn's guarantee. **Please don't!**

Questions?

Problems?

Suggestions?

Call 1-800-336-1414

Powderhorn Industries, 931 North Park Ave., P.O. Box 1443, Montrose, CO 81402